OM-DH0910EN 000

OPERATION MANUAL

General Cutting Straight

Ti-Max X **X65L**

S-Max M **M65**

FX **FX65 / FX65K**

EX **EX-5B**

Rx Only

MADE IN JAPAN

1 User and Indications for Use

User: Qualified Professionals (Dentist/Dental hygienist)

Indications for Use:

X65L/M65

General Cutting Straight is intended for the following application(s): Caries removal, cavity and crown preparation, removal of dental restorations (fillings and prostheses), finishing and polishing of teeth and dental restorations.

FX65/FX65K/EX-5B

General Cutting Straight is intended for the following application(s): Caries removal, cavity and crown preparation, removal of dental restorations (fillings and prostheses), finishing and polishing of teeth and dental restorations. Using a prophylaxis angle, prophylaxis treatment of the surface of teeth and dental restorations.

See the “12. Symbol” section on page 32 for reference.

2 Precautions for Handling and Operation

- Please read these precautions carefully and use only as intended or instructed.
- Safety instructions are intended to avoid potential hazards that could result in personal injury or damage to the device. Safety instructions are classified as follows in accordance with the seriousness of the risk.

Class	Degree of Risk
 WARNING	Hazard that could result in serious injury or damage to the device if the safety instructions are not correctly followed.

⚠ CAUTION	Hazard that could result in light or moderate injury or damage to the device if the safety instructions are not correctly followed.
NOTICE	General product specification information highlighted to avoid product malfunction and performance reduction.

⚠ WARNING

- The handpiece is delivered in a non-sterile condition and must be cleaned and sterilized prior to its first use and after each patient use.
- Clean and lubricate the handpiece immediately (within 1 hour) after each treatment to remove residue. Failure to properly maintain the handpiece may cause infection, product failure or overheating, leading to burn injuries. (Refer to “7. Post-use Maintenance”)
- If blood infiltrates inside a handpiece, clean and lubricate the handpiece immediately using the PANA SPRAY Plus or clean the handpiece using a washer-disinfector.

WARNING

- Cool down the cutting site with a dental air/ water syringe or X65L/M65 cooling spray while using the handpiece. Insufficient cooling may lead to overheating of the cutting site, causing burn injuries.
- Use polishing paste only when a prophylaxis angle is mounted on FX65/FX65K/EX-5B. Otherwise the polishing paste could enter the handpiece, causing failures of the ball bearing or burn injuries due to the overheating of the handpiece head.
- Keep any debris or other foreign materials away from inside the gear or the handpiece. Foreign materials remaining inside may lead to overheating, causing burn injuries or other accidents.

WARNING

- Ball bearings are wear-and-tear items and may require replacement. Be sure to inspect the handpiece before use (Refer to “6. Checking the Handpiece Before Each Use”). If abnormal vibration or noise is found, the bearings or other internal parts may be worn out or damaged. Damaged parts may lead to handpiece overheating causing burn injuries. If overheating is suspected, stop using the handpiece immediately and contact your Authorized NSK Dealer.

CAUTION

- Read this operation manual before use to fully understand the product functions and file for future reference.
- When operating the product, always consider the safety of the patient.
- Users are responsible for the operational control, maintenance and continual inspection of this product.
- Prior to clinical use, inspect the handpiece. Check for vibration, noise and overheating. If any abnormalities are found, stop using the handpiece immediately and contact your Authorized NSK Dealer. (Refer to “6. Checking the Handpiece Before Each Use”)
- Do not disassemble or alter the handpiece except as recommended by NSK in this operation manual.

CAUTION

- Do not allow any impact to the product. Do not drop the product. Deformation may cause the handpiece to fail during use.
- Operators and all others in the area must wear eye protection and a mask when operating this handpiece.
- Do not use burs with problems listed below as the bur may break, seize up or disengage from the chuck.
 - Bent, deformed, worn, rusted, broken, deficient bur.
 - Bur which is cracked on the edge or axis.
 - Non-ISO standard, or tampered bur.
- Do not use burs other than specified in “11. Specifications” as such burs may be accidentally released or break during rotation.

CAUTION

- Always keep the bur shank clean. Dirt or debris in the chuck could cause poor bur concentricity or low chuck retention force.
- Always insert the bur all the way into the chuck. If insertion is insufficient, premature failure of the bearings or accidental release of the bur may occur.
- Do not exceed Max. Bur Length shown in “11. Specifications”.
- Always follow the instructions provided by the bur manufacturer.
- Do not exceed the bur speed recommended by the bur manufacturer.
- Should the handpiece function abnormally during use, stop using the handpiece immediately and contact your Authorized NSK Dealer.

CAUTION

- Do not use the following fluids to wipe, immerse or clean the product; strong/super acid water, strong acid/alkaline chemicals, chlorine-containing solutions, solvents such as benzene or thinner.
- Perform regular function and maintenance checks. (Refer to “9. Periodical Maintenance Checks”)
- To avoid clinical downtime it is recommended that a spare be kept on hand in case of a breakdown during treatment.
- The operation of the handpiece is permitted only on dental units, which correspond to the standards IEC 60601-1 and IEC 60601-1-2.
- U.S. Federal law restricts this device to sale by or on the order of a licensed physician.

NOTICE

- Repairs of this product are only to be performed by authorized service technicians according to NSK requirements. Contact your Authorized NSK Dealer if repairs are necessary.

3 Package Contents

No.	Part Name	Quantity
1	E-Type Spray Nozzle	1
2	Bur Stopper *	1
3	Bur Stopper Puller *	1

*Included in the package for X65L / M65

CAUTION

- E-Type Spray Nozzle cannot be sterilized.

4 Connecting and Disconnecting the Handpiece from the Motor

4-1 Connecting

- 1 Insert the handpiece directly into the motor (Fig. 1).

When connecting an optic handpiece, twist the handpiece until it clicks and locks into position.

- 2 Confirm that the handpiece is firmly connected to the motor.

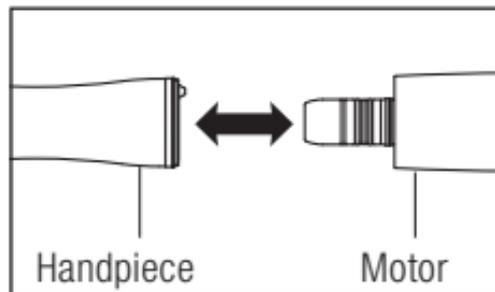
Fig. 1

4-2 Disconnecting

Hold the motor and the handpiece, then pull apart. (Fig. 1)

⚠ CAUTION

- Do not connect or disconnect the handpiece until the motor has completely stopped.
- Do not exceed the Max. Rotation Speed (Motor) shown in "11. Specifications".
- Connect ONLY to E type motors (ISO 3964).
- Make sure that the motor and handpiece are securely connected. The pressure of handpiece cooling air from the motor may cause the handpiece to jump out of the motor, which could result in injury.

5 Mounting and Removing the Bur

5-1 HP Bur

5-1-1 Mounting the Bur

- 1 Turn the bur lock ring in the direction of the “” or “O” mark until it clicks. (Fig. 2 or Fig. 3)
- 2 Insert the bur fully into the chuck then turn the bur lock ring in the direction of the “” or “C” mark until the “” or “C” mark and the “•” mark align with a click. (Fig. 2 or Fig. 3)
- 3 Confirm that the bur is secure by gently pulling and pushing the bur.

5-1-2 Removing the Bur

Turn the bur lock ring in the direction of the “” or “O” mark until it clicks then remove the bur. (Fig. 2 or Fig. 3)

X65L / M65

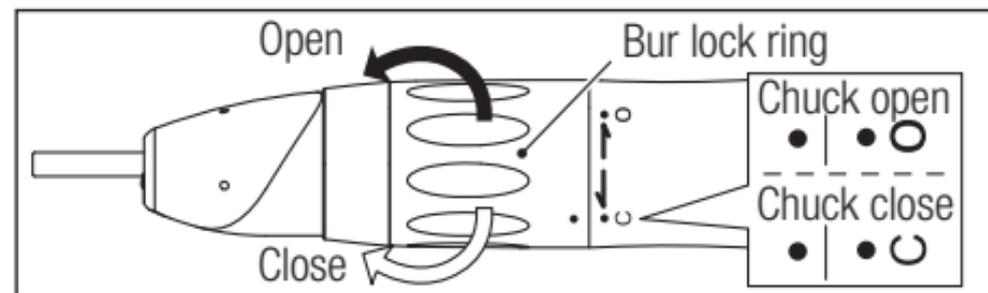
Fig. 2

FX65 / FX65K / EX-5B

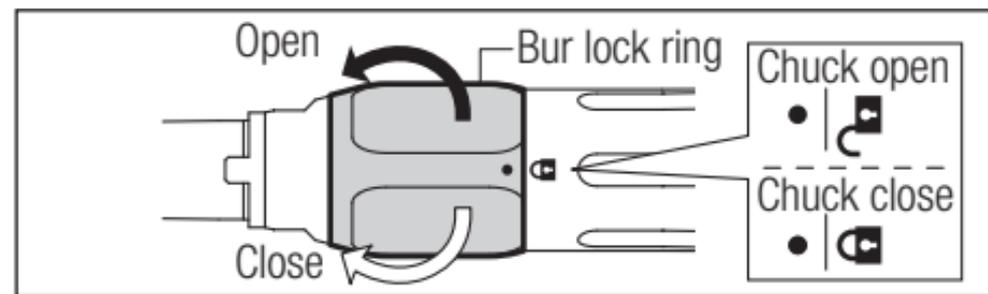
Fig. 3

5-2 CA Bur (X65L / M65)

5-2-1 Mounting the Bur

- 1 Turn the bur lock ring in the direction of the " O " mark until it clicks. (Fig. 2)
- 2 Insert the supplied Bur Stopper into the chuck. (Fig. 4)
- 3 Insert the CA bur until it stops.
- 4 Insert the bur fully into the chuck then turn the bur lock ring in the direction of the " C " mark until the " C " and the " ● " mark align with a click. (Fig. 2)
- 5 Confirm that the bur is secure by gently pulling and pushing the bur.

5-2-2 Removing the Bur

Turn the bur lock ring in the direction of the " O " mark until it clicks then remove the bur. (Fig. 2)

5-2-3 Removing the Bur Stopper

- 1 Turn the bur lock ring in the direction of the " O " mark until it clicks. (Fig. 2)
- 2 Insert the supplied Bur Stopper Puller into the chuck. Pull out the Bur Stopper by hooking at the slit of Bur Stopper. (Fig. 4, Fig. 5)

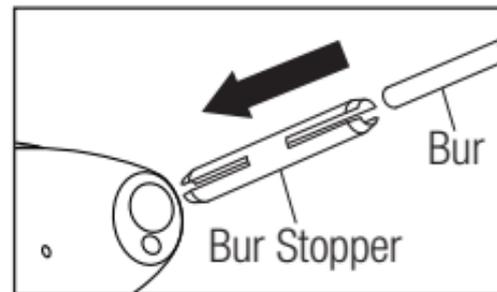
Fig. 4

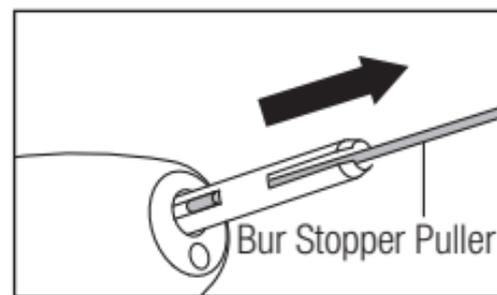
Fig. 5

5-3 Prophy angle (FX65 / FX65K / EX-5B)

5-3-1 Mounting the Prophy angle

- 1 Turn the bur lock ring in the direction of the “🔓” mark until it clicks.
- 2 Align the prophy angle with the groove of the handpiece and insert it deep into the chuck. (Fig. 6)
- 3 Turn the bur lock ring in the direction of the “🔒” mark until the “🔒” mark and the “●” mark align with a click.
- 4 Confirm that the prophy handpiece is secure by gently pulling and pushing the prophy angle.

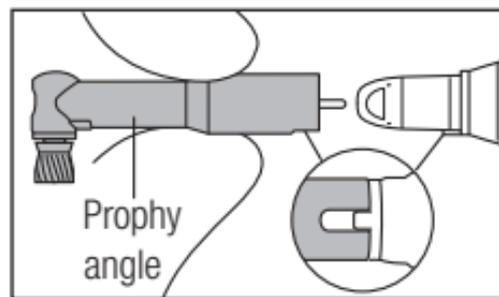
Fig. 6

5-3-2 Removing the Prophy angle

Turn the bur lock ring in the direction of the “🔓” mark until it clicks then remove the prophy angle.

⚠️ WARNING

- Always use a clean, sterile bur. Be sure to wear sterile gloves before mounting the bur.
- Do not rotate the motor when the bur lock ring is at the OPEN position or when a bur/prophy angle is not mounted in the chuck. In either situation, it may cause burn injuries due to sudden heat generation. In addition, the motor insert could be damaged which may lead to difficulty in connecting and disconnecting the handpiece.

WARNING

- After mounting the bur/prophy angle in the chuck, be sure to confirm that the “ ● ” mark and the “  ” mark align. If these marks are not completely aligned, burn injuries may occur due to handpiece overheating during rotation.
- When changing the bur during treatment, remove the bur and wipe around the bur insertion hole with a dry lint free cloth until it is visually clean, then mount a clean and sterile bur. Residual blood or debris may enter the handpiece while changing burs, which could lead to higher risk of cross-contamination.

CAUTION

- Do not use burs with problems listed below as the bur may break, seize up or disengage from the chuck.
 - Bent, deformed, worn, rusted, broken, deficient bur.
 - Bur which is cracked on the edge or axis.
 - Non-ISO standard, or tampered bur.
- Do not use burs other than specified in “11. Specifications” as such burs may be accidentally released or break during rotation.
- Insert a sterilized bur and always keep the bur shank clean. When changing burs, make sure that there is no dirt or debris in and around the bur insertion hole of the handpiece. Debris in the chuck could cause infection, heating, poor bur concentricity or low chuck retention force.

⚠ CAUTION

- Always insert the bur all the way into the chuck. If insertion is insufficient, premature failure of the bearings or accidental release of the bur may occur.
- Always follow the instructions provided by the bur manufacturer.
- Do not exceed the bur speed recommended by the bur manufacturer.
- Do not mount or remove the bur until the motor has completely stopped.
- Do not apply excess pressure to the bur as it may break or bend or become difficult to remove.

⚠ CAUTION

X65L / M65

- Mount the Bur Stopper firmly into the handpiece until it stops. If it is inserted incorrectly, it may be difficult to remove. The bur may jump out due to loose chuck, which could result in injury.
- When using the CA bur, make sure that the Bur Stopper is set inside the handpiece before installing it. Otherwise, the CA bur may get stuck inside and cannot be removed.
- Do not use the HP bur and Bur Stopper together. Otherwise, the bur may not be gripped properly and may jump out, which could result in injury.
- Do not use short shank CA bur. Otherwise, the bur may jump out due to loose chuck, which could result in injury.

6 Checking the Handpiece Before Each Use

Follow the check below before use. If any abnormalities are found, stop using the handpiece immediately and contact your Authorized NSK Dealer.

- 1 Ensure the bur and handpiece are securely connected. (Refer to “5. Mounting and Removing the Bur”.)
- 2 Rotate the handpiece for about one minute at the Max. rotation speed of the attached motor. During rotation, check for abnormalities such as abnormal rotation, vibration, noise.
- 3 For X65L/M65, check that cooling spray is supplied to the cutting portion of the bur.

- 4 After the handpiece rotation has completely stopped, touch the tip and bur lock ring of the handpiece to confirm they are not heating abnormally (Fig. 7).

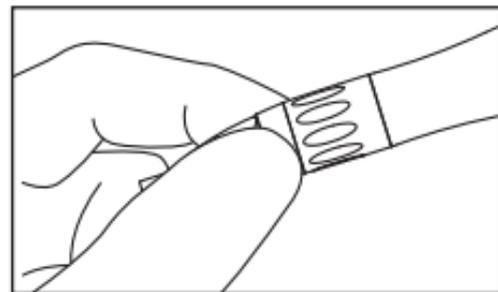
Fig. 7

⚠ CAUTION

- To avoid injury, keep your hands away from the bur during rotation.

7 Post-use Maintenance

After each patient, maintain the product as follows.

⚠ WARNING

- To ensure cleaning and sterilization efficacy, use only the following procedures for reprocessing.
- Follow any additional local directives, standards, and guidelines for cleaning and sterilization.
- The handpiece is delivered in a non-sterile condition and must be cleaned and sterilized prior to its first use and after each patient use.
- Clean and lubricate the handpiece immediately (within 1 hour) after each treatment to remove residue. Failure to properly maintain the handpiece may cause infection, product failure, overheating leading to burn injuries.

7-1 Cleaning at point-of use

⚠ CAUTION

- Do not use the following liquids to wipe, immerse, or clean the product: strong/super acid water, strong acid/alkaline chemicals, chlorine-containing solutions, solvents such as benzine or thinner.
- Do not immerse NSK instruments in disinfectant solution or clean in ultrasonic devices.

- 1 Always wear protective gloves, a mask, and protective goggles for safety purposes and to minimize the risk of infection. (Fig. 8)

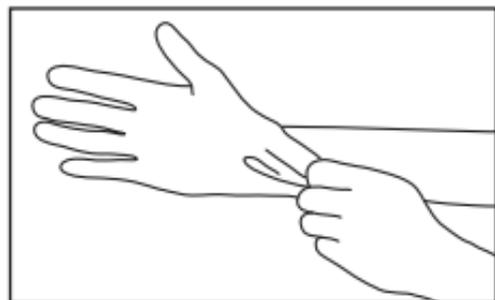
Fig. 8

- ② Operate the handpiece at the chair-side for at least 20 seconds to purge fluids in X65L and M65 (Fig. 9). After the handpiece rotation has completely stopped, touch the tip and bur lock ring of the handpiece to confirm they are not heating abnormally.

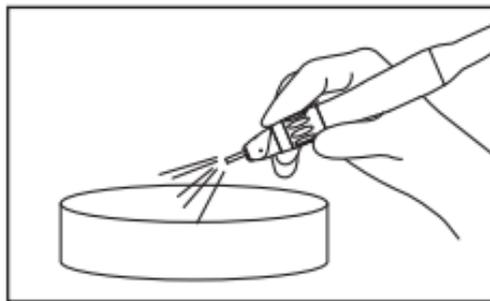
Fig. 9

⚠ CAUTION

- If the tip of the handpiece is heating abnormally, stop using the handpiece immediately and contact your Authorized NSK Dealer.
- Be sure to take the step “7-1 Cleaning at point-of use ②” before removing the bur. If the handpiece operates without the bur, it may cause burn injuries due to sudden heat generation. In addition, the motor insert could be damaged which may lead to difficulty in connecting and disconnecting the handpiece.

- 3** Remove the bur. (Fig. 10)
When removing the CA bur from X65L and M65, also remove the Bur Stopper using the Bur Stopper Puller. Refer to “5-2-3 Removing the Bur Stopper” for details.

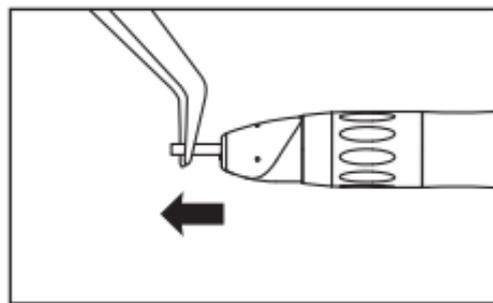
Fig. 10

When removing the prophy angle from FX65, FX65K and EX-5B, refer to “5-3-2 Removing the Prophy angle” for details.

- 4** Wipe the exterior of the handpiece clean using a cloth moistened with disinfectant or ethanol with a concentration of 60-90%. (Fig. 11)

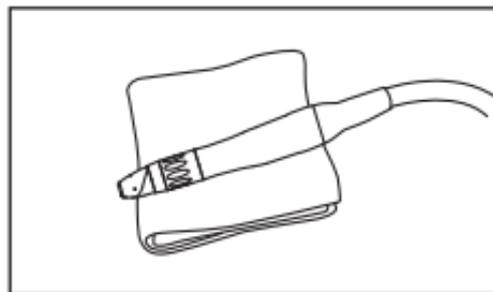
Fig. 11

When using disinfectant, follow the instructions given by the manufacturer of the disinfectant.

Use a state-sanctioned disinfectant with proven bactericidal, fungicidal, and virucidal properties. The following disinfectants can be used in the United States and Canada.

CaviCide®, CaviWipes® (manufactured by Metrex)

- 5** Remove the handpiece from the motor. (Fig. 12)
6 Carry the handpiece to the decontamination area.

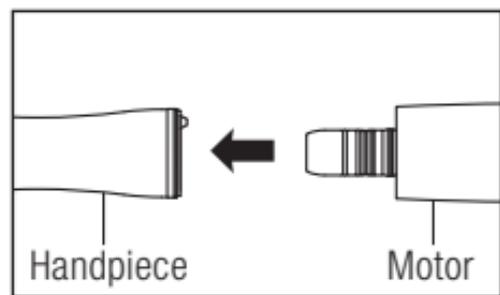
Fig. 12

7-2 Cleaning and Drying

Perform cleaning and drying using either the manual or automatic method.

Manual Method (Cleaning and Drying the Exterior)

Wash the handpiece, Bur Stopper and Bur Stopper Puller under running water following the procedures below.

(Water condition: $\leq 38^{\circ}\text{C}$ (101°F), $\geq 3.5\text{L}/\text{min}$; water should be of the same quality as drinking water)

- 1 Clean the external surfaces of the handpiece with a soft bristled toothbrush for 15 seconds or more. (Fig. 13)

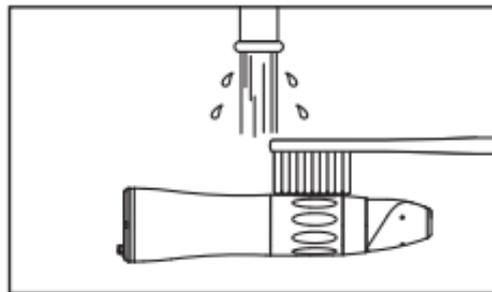
Fig. 13

- 2 Clean around the bur insertion hole for 15 seconds or more, using an interdental brush (wire thickness 0.7mm) that conforms to size 4 in ISO 16409. (Fig. 14)
- 3 Clean the external surface of the Bur Stopper and Bur Stopper Puller with a soft bristled toothbrush for 15 seconds or more. (Fig. 15)

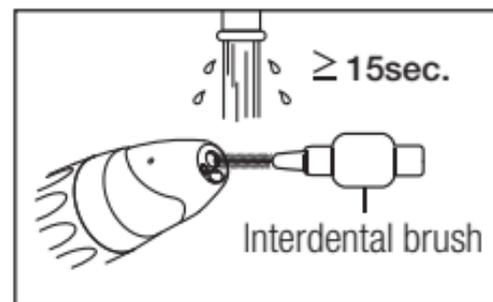
Fig. 14

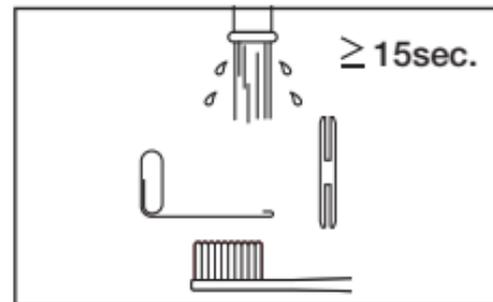
Fig. 15

Under appropriate lighting (natural office lighting, 500 lx or higher), inspect the handpiece, Bur Stopper and Bur Stopper Puller for blood or other debris. If any visible debris remains, repeat the process until the handpiece, Bur Stopper and Bur Stopper Puller are visually clean.

Optional

Wipe the exterior of the handpiece, Bur Stopper and Bur Stopper Puller clean using a cloth moistened with disinfectant or ethanol with a concentration of 60-90%. (Fig. 16)

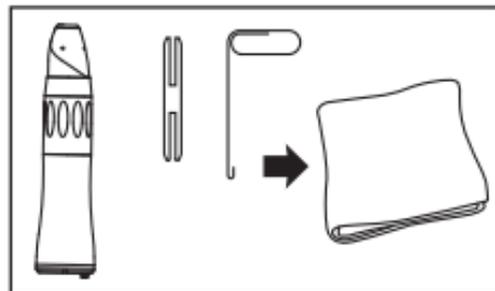
Fig. 16

NOTICE

- Wiping is not an alternative method of cleaning. Please do not skip the required cleaning steps which are previously displayed.

When using disinfectant, follow the instructions given by the manufacturer of the disinfectant.

Use a state-sanctioned disinfectant with proven bactericidal, fungicidal, and virucidal properties.

The following disinfectants can be used in the United States and Canada.

CaviCide[®], CaviWipes[®] (manufactured by Metrex)

After cleaning, if residual moisture is present, wipe off with dry cloth or blow it off with compressed air ($\leq 0.35\text{MPa}$) until there is no moisture in the interior and exterior.

When blowing off with compressed air, cover the handpiece with cloth to prevent scattering of water.
Proceed to “7–3 Cleaning and Lubrication”.

Automatic Method (Cleaning and Drying the Exterior and Interior)

NSK handpieces with this symbol are compatible with medical washer-disinfectors.

* FX65, FX65K and EX-5B cannot be cleaned and disinfected with a washer-disinfector.

Due to the variation in cleaning/disinfection cycles and cleaning agents available from the various manufacturers, NSK has validated the Getinge 46 Series Washer Disinfector Cycle P7-Anesthesia for automatic processing/reprocessing of the product.

Only validated procedures must be used for cleaning and sterilization. When using reprocessing procedures that are different from those described in this manual, those procedures must be validated by the respective practice or hospital using a washer-disinfector that conforms to ISO 15883-1.

NOTICE

- All water used in cleaning is tap water of drinking water quality unless otherwise indicated.

- 1 Prepare and pre-clean the device as described in step “7-1 Cleaning at point-of use”.
- 2 Wipe off a dirt on a part held by a handpiece holder using the cloth moistened with 60-90% ethanol or disinfectant.
When using disinfectant, follow the instructions given by the manufacturer of the disinfectant.
Use a state-sanctioned disinfectant with proven bactericidal, fungicidal, and virucidal properties.
The following disinfectants can be used in the United States and Canada.
CaviCide[®], CaviWipes[®] (manufactured by Metrex)

- 3 Perform main wash under the following conditions. Use a Getinge 46 Series washer-disinfector (or equivalent device):
Pre-Wash 1: Cold water (<43°C (110°F)) for 2 min
Pre-Wash 2: None - Not Applicable
Enzyme Wash: Hot tap water and 60ml of heated (35°C (95°F)) neutral enzymatic detergent for 3 min at 49°C (120°F)
Detergent Wash: None - Not Applicable
Rinse 1: Hot tap water for 2 min
Rinse 2: Hot tap water for 2 min
RO Final Rinse: Hot tap water for 32 min at 75°C (167°F)
Drying: 20 min
Upon removal from the washer-disinfector, if the residual moisture is present, wipe off with dry cloth or blow it off with compressed air ($\leq 0.35\text{MPa}$) until there is no moisture in the interior and exterior.

- 4 After cleaning and drying the handpiece, confirm that it is clean under appropriate lighting (500 lx or higher). If dirt still remains, repeat the procedures.
- 5 Proceed to “7–3 Cleaning and Lubrication”.

⚠ CAUTION

- Handpieces must be removed from the washer-disinfector immediately (within 1 hour) after the cleaning and drying cycle is complete to prevent corrosion.
- After washing with washer-disinfector, prior to lubrication, dry the product until all internal moisture is completely removed. Washer-disinfector moisture remaining inside the product could reduce the effect of lubrication and could cause corrosion inside of the product.

7–3 Cleaning and Lubrication

- 1 Attach an E-Type Spray Nozzle to the nozzle of the PANA SPRAY Plus.
- 2 Shake the can 3 or 4 times, and insert the E-Type Spray Nozzle into the rear of the handpiece while holding the tip of the handpiece with a cloth etc.
- 3 Hold the handpiece, and spray for 2 or 3 seconds until oil comes out of the tip (Fig. 17). Repeat lubrication until foreign material stops coming out of the tip.

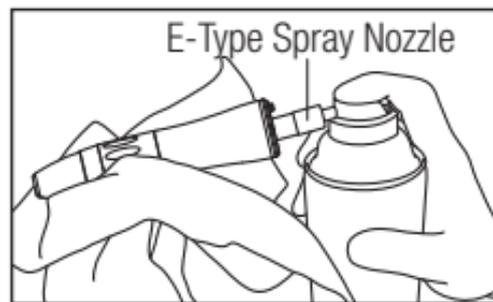
Fig. 17

⚠ WARNING

- If blood infiltrates inside a handpiece, clean and lubricate the handpiece immediately using the PANA SPRAY Plus or clean the handpiece using a washer-disinfector.

⚠ CAUTION

- Do not use sprays other than NSK PANA SPRAY Plus. Using sprays other than PANA SPRAY Plus may cause the handpiece to overheat.
- Hold the spray can upright.
- Firmly hold the handpiece to prevent it slipping when spray pressure is applied.
- Spray lubricant until it expels from the tip of the handpiece.

NOTICE

- NSK recommends the use of “Spray Mist Absorber” (**REF** Y900084) to prevent oil mist expelling out of the tip of the handpiece.

7-4 Packaging, Sterilizing, and Drying

- 1 Insert the handpiece, Bur Stopper and Bur Stopper Puller into an FDA-approved sterilization pouch(es) that conforms to ISO 11607-1, and seal the pouch(es).
- 2 Perform steam sterilization with the following conditions.

Type	Gravity Displacement	Pre-Vacuum (Dynamic Air Removal)
Temperature	132°C (270°F)	132°C (270°F)
Full Cycle Time	15 min	4 min
Drying Time	30 min	30 min

CAUTION

- Use an FDA-approved steam sterilizer to perform sterilization.
- Follow local rules, regulations, and guidelines regarding the reprocessing of devices.
- Do not touch the product immediately after steam sterilization as it will be very hot and must remain in a sterile condition.
- Place each product in an individual sterilization pouch for steam sterilization. This is to prevent possible discoloration and damage to the product due to chemical residue from other instruments.
- Clean and lubricate the handpiece prior to sterilization. If blood remains on the internal surface it can become clotted and cause product failure.

CAUTION

- Do not heat or cool the product too quickly. Rapid change in temperature could cause damage to the product.
- Be sure to use sterilizers that can perform sterilization up to 135°C (275°F). In some sterilizers, the chamber temperature may exceed 135°C (275°F). Do not use these sterilizers as failure of the handpiece/Bur Stopper/Bur Stopper Puller could occur. Contact the sterilizer manufacturer for detailed information about cycle temperatures.
- Steam sterilization is recommended for the product. The validity of other sterilization methods (such as plasma sterilization or EOG sterilization) is not confirmed.

7-5 Storage

Store the product in a dry, clean location.

CAUTION

- After the sterilization and drying cycles are complete, remove the handpiece immediately from the sterilizer to store it.
- Store the product in a well ventilated place out of direct sunlight and within the range of temperature, humidity and pressure specified in “11. Specifications”.
- Sterilization is not guaranteed after the sterilization retention period specified by the manufacturer and seller of the sterilization pouch has elapsed. If the sterilization retention period has elapsed, perform sterilization again with a new sterilization pouch.

8 Cleaning of Optic Illumination Points

In case of X65L, when debris is attached to optic illumination points (glass rod), wipe clean all the optic illumination points using an alcohol-immersed cotton swab. Remove all debris. (Fig. 18)

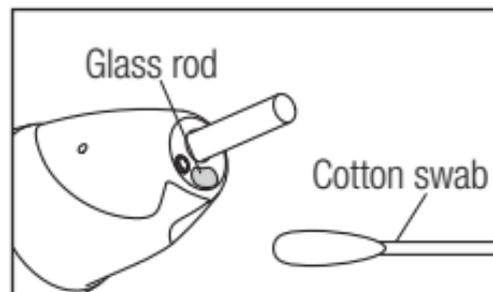
Fig. 18

⚠ CAUTION

- Do not use a sharp tool to clean the glass rod. It could damage the glass and reduce the light transmission.

9 Periodical Maintenance Checks

Perform periodical maintenance checks every three months, referring to the check sheet below. If any abnormalities are found, contact your Authorized NSK Dealer.

Points to check

Details

Points to check	Details
Rotation	Rotate the handpiece and check for abnormalities such as abnormal rotation, vibration, noise, and overheating.
Connection	Check the connection between the motor, bur and handpiece for any problems.

10 After-sales Service

10-1 Warranty

NSK products are warranted against manufacturing errors and defects in materials. NSK reserves the right to analyze and determine the cause of any problem. Warranty is voided should the product be not used correctly or for the intended purpose or has been tampered with by unqualified personnel or has had non NSK parts installed. Replacement parts are available for seven years beyond discontinuation of the model. Contact your Authorized NSK Dealer if repairs are necessary.

10-2 Accessories

Model	REF	Applicable Handpiece Series and Models
E-Type Spray Nozzle	Z019090	All models within each handpiece series
Bur Stopper	H273124	<u>Ti-Max X</u> X65L <u>S-Max M</u> M65
Bur Stopper Puller	H273701	<u>Ti-Max X</u> X65L <u>S-Max M</u> M65

10-3 Option List

Model	REF	Handpiece Series and Model	Remarks
PANA SPRAY Plus	Z182600	All models within each handpiece series	-

11 Specifications

Model	X65L	M65
Handpiece Connection Type	ISO 3964 Type3	ISO 3964 Type2
Max. Rotation Speed (Motor)	40,000min ⁻¹	
Max. Rotation Speed (Handpiece)	40,000min ⁻¹	
Gear Ratio	1:1 Direct Drive	
Bur Type	ISO 1797 Type2 Ø2.334-2.35mm HP Bur Standard shank ISO 1797 Type1 Ø2.334-2.35mm CA Bur Standard shank (With Bur Stopper)	
Chuck Length	11.5mm (CA Bur) / 29.5mm (HP Bur)	
Max. Bur Length	22.5mm (CA Bur) / 44.5mm (HP Bur)	
Max. Working Part Diameter	Ø4.0mm	
Optic	Glass rod	Non-optic

Model	X65L	M65
Water Spray Type	Internal Spray	
Coolant Water Pressure	0.2MPa (2.0kgf/cm ²)	
Chip Air Pressure	0.2MPa (2.0kgf/cm ²)	
Water Supply	≥50mL/min (0.2MPa)	
Chip Air Supply	≥1.5L/min (0.2MPa)	
Compatible with washer-disinfector	Yes	

Model	FX65	FX65K*	EX-5B
Handpiece Connection Type	ISO 3964 Type1		
Max. Rotation Speed (Motor)	40,000min ⁻¹		
Max. Rotation Speed (Handpiece)	40,000min ⁻¹		10,000min ⁻¹
Gear Ratio	1:1 Direct Drive		4:1 Reduction
Bur Type	ISO 1797 Type2 Ø2.334-2.35mm HP Bur Standard shank / ANSI/ADA No.85-1 Disposable Prophy angle /ISO 14457 Prophy handpiece)		
Chuck Length	30mm		
Max. Bur Length	44.5mm		
Max. Working Part Diameter	Ø4.0mm		
Optic	Non-optic		
Compatible with washer-disinfector	No		

*This product can be connected to a motor with Ø16mm outer diameter.

Use Environment	Temperature: 10 - 35°C, Humidity: 30 - 75% (No Condensation)
Transportation and Store Environment	Temperature: -10 - 50°C, Humidity: 10 - 85%, Pressure: 500 - 1,060hPa

12 Symbol

Open

Close

Position

This product can be sterilized in a steam sterilizer at 135°C (275 °F).

The product can be used with a washer-disinfector for thermal disinfection.

Conforms to CE European Directive of "Medical device directive 93/42/EEC."

Manufacturer.

Date of manufacture

Caution: U.S. Federal law restricts this device to sale by or on the order of a licensed physician.

GS1 DataMatrix for Unique Device Identifier.

See operation manual.

Caution, Consult operation instructions.

Temperature limitation

Humidity limitation

Atmospheric pressure limitation

UDI

Unique device identification

REF

Catalog number (Order code)

SN

Serial No.

CAN

Device Name for Health Canada.

13 Disposing Product

In order to avoid the health risks of operators handling the disposal of medical equipment, as well as the risks of environmental contamination caused thereof, a surgeon or a dentist is required to confirm the equipment is sterile. Ask specialist firms who are licensed to dispose of specially controlled medical wastes, to dispose the product for you.

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NSK America Corp. www.nskdental.com